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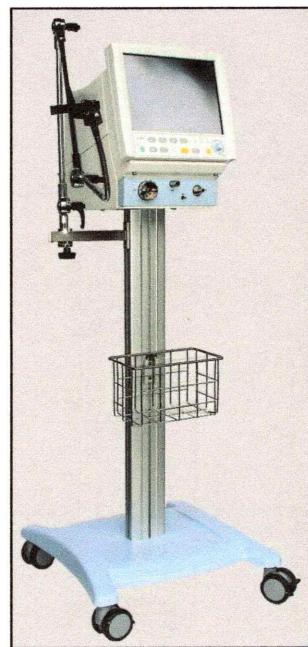
Phone: 02603 9600-0
Fax: 02603 9600-50
e-mail: www.info@hul.de

8. Index

- About this user manual 4
- Accessories 7, 10
- Accessories and consumables 7
- Additional information 5
- Approved cleaning and disinfection agents 11
- Assembly instructions 5
- Configurations 9
- Defect 12
- Dimensions and weight 8
- Disposal of defect parts 12
- Floor assembly 8, 9, 10
- Further documentation 5
- General instructions 10, 11
- General safety instructions 6
- Humidifier 10
- Important topics in the user manual 4
- Index 14
- Information and warnings 4
- Information on dangers 5, 10
- Intended use 6
- Intervals for maintenance 12
- Maintenance and correct operating condition 6
- Maintenance and repair 12
- Nameplate 13
- Operating Data 8
- Preparation, cleaning, sterilisation 11
- Releases 10
- Safe working with the stand 7
- Scope of delivery 9
- Storage of documentation 5
- Structure and purpose of user manual 4
- Suitable installation and ambient conditions 7
- Support arm 10
- Symbols 9
- Technical data 8
- Technical data and equipment 8
- Ventilator 6, 10

6. Maintenance and repair

Intervals for maintenance

**DANGER**

Maintenance, repair and return of contaminated devices

Danger of infection!

- Clean and disinfect the device or device components before every maintenance procedure – also when returning the device for repair!

**CAUTION**

Incorrect maintenance

Danger of malfunction and damage to the device!

- Only technicians trained by the manufacturer are authorized to carry out the service and safety checks.
- Suitable instruments and test equipment are required.

Every 12 months	Next service:
<p>The following work is required:</p> <ul style="list-style-type: none">▪ Check for damages▪ check all loose bolted connections▪ Wheels free moving	

Disposal of defect parts

- Defective parts generally appear during servicing, and are disposed by the service technician.
Otherwise these materials must be disposed of in accordance with regulations. If in doubt, refer to the hygiene guidelines of your hospital or contact a representative of Heinen + Löwenstein.

4. Releases**General instructions**

The stand is designed to carry ventilators with its accessories.

Because of its low undercarriage it can be moved very close to the patient bed.

**CAUTION**

Make sure to consult the separate user manuals of the used ventilators and accessories, also for information on dangers.

Releases

Ventilators The stand is released for the following ventilators:

- leoniplus (86 cm post height)
- leoni2 (86 cm post height)
- Elisa (78 cm post height)

accessories (optional) The stand is released for mounting the following humidifier:

- MR850
- PMH 5000
- HumiCare200

The stand is released for mounting the following support arms:

- support arms ($l_{max} = 150 \text{ mm}$)

**CAUTION**

For mounting the support arm
use only the rails released.

3. Technical data and equipment**Technical data****Dimensions and weight**

Dimensions / weight	
W x H x D (construction with long post and undercarriage)	52 cm x 100 cm x 52 cm
W x H x D (construction with short post and undercarriage)	52 cm x 92 cm x 52 cm
Weight	9 kg
Material	aluminum

Operating Data**Operating Data**

under 93/42/EEC Appendix IX

medical device class I

2. General safety instructions

The liability for proper function of the stand is irrevocably transferred to the owner or operator of the stand if it is serviced or repaired by persons who are not employed by *Heinen + Löwenstein GmbH* Service department or are not authorized by *Heinen + Löwenstein* or if it is operated in a manner contrary to its intended use.

Heinen + Löwenstein GmbH cannot be held responsible for damage caused by non-compliance with the above stipulations. The warranty and liability provisions of the terms of sale and delivery of *Heinen + Löwenstein GmbH* are also not modified by the following recommendations.

Intended use

Area of application Stand for ventilators

Proved operators The stand is operated by a physician or a person trained and qualified to operate the stand under the direction of a physician. Every user must be trained on the stand and must be familiar with the user manual and the operation of the stand.

Combination with other devices Combination with other devices that are not mentioned in the user manual is permitted only in consultation with the manufacturer.

Maintenance and correct operating condition

Maintenance The stand is classified as a medical device class I under the European Directive.

- The stand should be checked regularly and repairs if necessary should be carried out.
- Only technicians trained should do repairs.

Correct operating condition If a fault is detected or the function and security of the stand can not be assured it must not be used.

1. About this user manual

Important topics in the user manual

This user manual describes the components of the stand and its operation.

It includes:

- information about the stand
- information on safety measures that must be observed when working with the stand
- an overview of all stand components
- instructions for operating the stand
- information on service, cleaning and sterilization

Structure and purpose of user manual

Read the user manual carefully before starting to use the stand.

Later, when you are familiar with the basic operation of the stand, the user manual will be useful as a reference for answering more detailed questions. Use the table of contents and the index to find a topic quickly.

Information and warnings

Information

Information is provided to supplement the instructions for actions.

The information suggests actions that can make it easier and more efficient to operate the stand.

This page is intentionally left blank